Saliva Collection Kit

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In vitro diagnostic use only

INTENDED USE / GENERAL INFORMATION

The Saliva Collection Kit provides all components for saliva collection by a lay user and subsequent safe transport to a laboratory for diagnostic testing on SARS-CoV-2. After spitting in the empty tube, InActiv Blue™ must be added to make the sample non-infectious and to stabilize the RNA for at least 30 days. InActiv Blue™ is a virus-inactivating and lysis solution that abrogates the infectious potential of the collected material without affecting the integrity of the RNA to be used in a diagnostic test based on RT-qPCR or other molecular method. InActiv Blue™ contains guanidine thiocyanate and N-lauroylsarcosine for virus and RNase inactivation. Complete product composition is provided in the MSDS.

MATERIAL INCLUDED WITH THE KIT

Saliva Collection Kit (product code: IB_COL) containing:

- 1 x 6 ml PET tube filled with 2 ml InActiv Blue™ (product code: 456604)
- 1 x empty Saliva Collection Tube (product code: IB_COT)
- 1 x funnel
- 1 x Alcomed pad
- 1 x instructions for use leaflet

A certificate of analysis and MSDS are available upon request or can be downloaded from the website www.inactivblue.com

SPECIMEN TYPE

Saliva sample collected by spitting

PERFORMANCE PARAMETERS

- InActiv Blue™ is able to completely inactivate SARS-CoV-2 virus
- InActiv Blue™ maintains the stability of SARS-CoV-2 RNA for at least 30 days (2-25 °C).
- Robust formulation: proven stability after repeated freezing/thawing the sample and 8 days exposure at 37 °C

The whitepaper that further describes the safety and performance characteristics for the media is available on www.inactivblue.com

STORAGE/DISPOSAL

- Store reagents/samples between 2 °C and 25 °C.
- Keep away from direct (sun) light.
- Product can be used for 16 months after date of production.
- Do not use after expiry date.
- InActiv Blue™ needs to be disposed in accordance with local regulations for disposal of medical devices / hazardous substances. As indicated in the warning section below, contact between InActiv Blue™ and bleach must be avoided.

BIBLIOGRAPHY LITERATUR

1. Centers for Disease Control and Prevention. (2020, February 14). Coronavirus Disease 2019 (COVID-19): Guidelines for Clinical Specimens. Retrieved February 27, 2020, from <https://www.cdc.gov/coronavirus/2019-nCoV/lab/guidelines-clinical-specimens.html>

TECHNICAL SUPPORT TECHNISCHER SUPPORT

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At test laboratory:

Samples should be analyzed within 30 days upon sample collection. Invert tube 10 times to ensure that the complete inner surface of the tube has been in contact with the inactivating buffer solution. After this, samples should be centrifuged to avoid cross contamination when opening the tubes due to medium present in the cap. Use validated laboratory procedures for PCR testing. Known and demonstrated compatibility with test platforms can be found on www.inactivblue.com. InActiv Blue™ cannot be used with analytical procedures including a bleach decontamination step.

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DO NOT DECONTAMINATE TUBES OR LAB ENVIRONMENT WITH BLEACH!
InActiv Blue™ contains guanidine thiocyanate that in combination with hypochlorite may form harmful cyanide gas.

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Harmful if swallowed, causes severe skin burns and eye damage, harmful to aquatic life with long lasting effects.

All human, organic material should be considered potentially infectious. Handle all specimens as if capable of transmitting viruses. Always wear protective clothing when handling specimens and reagent (gloves, lab vest, surgical mask, eye/face protection).

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